

NoMoCo Pillow Support System Kit Cleaning Instructions: Healthcare Fabric

Washing and Disinfection

Spray clean with any hospital grade disinfectant (not to exceed 1:10 bleach solutions as recommended by the CDC for disinfecting) and wipe dry. Proprietary disinfectants may be used provided manufacturer's instructions are followed.

Avoid vent areas when spraying each pillow. Penetration into the foam vents could cause foam degradation. Use recommended drying methods to maximize the hospital grade disinfectant's effectiveness.

All cleaning agents, and disinfectants, must be thoroughly rinsed off and the item dried before storage. Failure to do this may result in the accumulation of reagent that could damage the polyurethane coating, react with the bed frame, or negate the biocompatibility results of the Dartex fabric.

Cover with drape if used in procedures where fluid contamination is possible and then spray with disinfectant and wipe clean with a soft cloth.

*Carry case is made of durable plastic but has small ferromagnetic hinges and should be stored outside the scanner room.